drive™

www.drivemedical.com

Rev.2.05.30.14

home bed assist handle

mango auxiliar
para la cama

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installation instructions

Weight Capacity: 250 lbs (113 Kg)

This bed assist rail is designed for use on a bed with a box spring and mattress.

1. Remove contents from carton.
2. Remove mattress from bed.
3. Attach the end cross brace to the base support frame. Replace the both locking collars. Be sure the locking pins go through the tubing and protrudes on the opposite side.
4. Attach the padded hand brace to the base support frame by depressing the brass lock pins and sliding the padded hand brace into the tubing until the lock pins “snap” through the positioning holes.
5. Attach legs by depressing brass lock pins and sliding the legs into the tubing until the lock pins “snap” through the positioning holes.
6. Place assembled base support frame on top of the box spring.
7. Locate a convenient distance from the head of the bed to position the bed assist rail.
8. Return mattress to bed.
9. After the bed rail is installed on the bed, remove the locking collars. Once removed, pull up or push down from the top of the handle to the desired height. Once the desired height is determined, making sure both holes line up, replace the both locking collars. Be sure the locking pins go through the tubing and protrudes on the opposite side.
10. Adjust legs so they rest on the floor by depressing push pins and setting to the desired height, making sure the push pins “snap” back into position.

lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.