




CAUTION:

Stability - Wheelchair is designed to be safe and stable as long as center of gravity is maintained while going about normal activities. Care must be taken while performing activities that shift your center of gravity in the chair. Never reach so far that require you to shift in the seat or bend over out of the chair. When you have to reach for objects that require you to lean, be certain front casters are pointing in a forward position that extends the wheel base and will stabilize the chair.

Ramps - Be sure of your own capabilities, limitations and strength before attempting to go up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. Assistance is recommended when going up or down inclines greater than 10°. If you must stop on an incline, avoid sudden, abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Attempting to use the wheel lock brakes could result in accidental locking, tipping to one side, or cause a sudden stop. Avoid changing direction when going down an incline, as this could result in instability of tipping over.

LIFETIME LIMITED WARRANTY

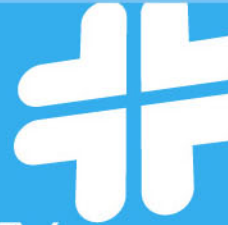

Your MERITS product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.


If you have a question about your MERITS device or this warranty, please contact an authorized MERITS dealer.

Distributed by:



<http://www.meritshealth.com>
 P.O. Box 150356, CAPE CORAL, FL 33915
 TEL: 1-800-9MERITS

© 2005 Merits Inc. All rights reserved.

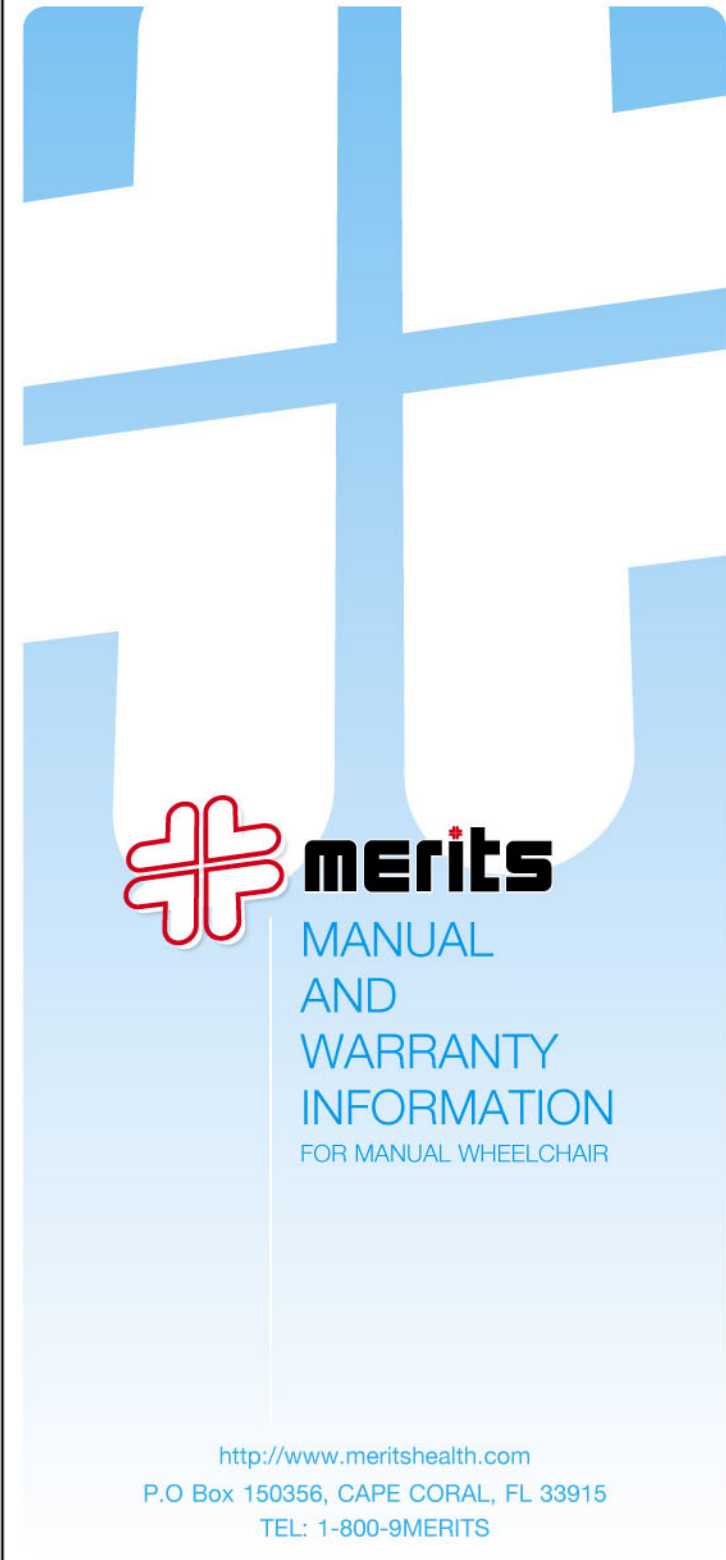

MANUAL AND WARRANTY INFORMATION FOR MANUAL WHEELCHAIR

<http://www.meritshealth.com>
 P.O. Box 150356, CAPE CORAL, FL 33915
 TEL: 1-800-9MERITS